

## Trifectiv® Plus Paediatric Care 100 mL (3.4 FL. OZ)

Instructions/package insert

**Trifectiv® Plus Paediatric Care** is ideal for treating skin conditions, infections, wounds, bites, scrapes, burns and other trauma to the skin. The active ingredient is hypochlorous acid (HOCl).

**Trifectiv® Plus Paediatric Care** helps to cleanse the area and accomplish the goals of wound bed preparation in a safe, effective, and natural way. This solution of hypochlorous acid (HOCl) is anti-bacterial, anti-fungal and antiviral and soothes the wound. This ensures better, faster wound healing with a lower risk of infection and a marked improvement in pain and odour in chronic non-healing wounds. The potential for scarring as the wound heals is also reduced.

### Composition

Sterile solution of hypochlorous acid 380 mg/L and water (Aqua)

### Medical device classification

Medical device Class C (ZA). Suitable for use on open wounds.

Wound irrigation solution, antimicrobial.

### Indications

Any condition where the skin's integrity is compromised or for skin infections.

### Contra indications

Expect diminished efficacy in the presence of cortisone and anti-inflammatory gel.

### Warning

If the condition does not improve within 48-96 hours, consult with your health care practitioner.

Do not use the solution if it is beyond the expiry date.

### Safety

The product is safe to use during pregnancy and lactation, as well as on children, babies and animals.

### QUICK GUIDE

1. Spray the area with **Trifectiv® Plus Paediatric Care** and allow to air dry for a few minutes.
2. Repeat frequently as required throughout the day.



### USAGE DIRECTIONS FOR SPECIFIC CONDITIONS

#### Fungal infections such as Cradle Cap

- Spray **Trifectiv® Plus Paediatric Care** onto the affected areas after cleaning and drying the area. Allow to dry naturally. Repeat at least twice per day.

*\*Note: infection should clear within two weeks, however it is important to continue use of **Trifectiv® Plus Paediatric Care** until the cradle cap has cleared up completely.*

#### Oral thrush

- Spray **Trifectiv® Plus Paediatric Care** directly into baby's mouth four times per day.
- Spray onto the nipples after each feed or at least four times per day. Remove any excess milk from the area with a tissue before spraying. Allow to dry naturally.

#### Nappy rash

- Clean nappy area with warm water or wet wipes. Do not use soap.
- Gently pat dry with a clean dry towel. Once dry, spray the affected area with **Trifectiv® Plus Paediatric Care** and allow to air dry completely.
- Once the skin is dry apply a barrier cream to the skin. Do not use an ointment such as petroleum jelly.
- Repeat above steps with every nappy change or at least once every 2 hours until the nappy rash has healed.

#### Inflamed or infectious skin conditions, sunburn, bites

- Spray **Trifectiv® Plus Paediatric Care** 4 x per day over the area and allow to dry naturally.

#### Cuts, scrapes, grazes

- Wounds should be cleaned prior to application of the first dressing of **Trifectiv® Plus Paediatric Care** being applied. Do this by washing the

wound with a bath sponge, warm water and a little bubble bath or liquid cleanser.

- Ensure that no cleanser remains on the wound after rinsing with clean water (do not use saline).
- Pat the wound dry with a clean towel. No further cleaning of the wound will be necessary when using **Trifectiv® Plus Paediatric Care**.
- Thoroughly spray the affected area with **Trifectiv® Plus Paediatric Care** and allow to dry naturally. Do not rub the solution into the skin.
- Continue to spray the wound 3 to 4 times a day. If necessary, you may cover the wound with a breathable dressing.

#### Burns

- Remove any dead skin or other debris.
- Spray **Trifectiv® Plus Paediatric Care** onto the affected area.
- Apply a single layer of **non-medicated paraffin gauze** to the wound to minimise discomfort during dressing changes.
- Wet 3-5 layers of clean non-sterile woven gauze with 5 mL\* of **Trifectiv® Plus Paediatric Care** and apply over the paraffin gauze. Make sure that the entire area is covered.
- Secure with a crepe bandage or breathable dressing.
- Change the dressing daily following the steps above.

#### Side effects and special precautions

Slight stinging may occur for the first day of use.

**Do not use with waterproof**, silver-impregnated or medicated dressing.

Do not mix into or use with or after disinfectants like saline, chlorhexidine, iodine, or any other branded disinfectant.

In some cases, additional sharp debridement of visible necrotic tissue may be required before the first application of **Trifectiv® Plus Paediatric Care**, for optimal results. Please consult with a medical professional if necessary.

*THE BOTTLE IS GLASS please take care not to drop it.*

*Do not leave within the reach of babies and toddlers when unattended.*

*Replace the cap after use to protect the nozzle and prevent breakage.*

#### Known symptoms of overdose

None known

#### Identification

Clear, colourless fluid

#### Presentation

100 mL (3.4 FL OZ) in a white **GLASS** bottle with atomiser pump.

#### Storage instructions

Store at room temperature.

Protect from direct sunlight.

Discard any unused product 12 months after opening.

#### Name and address of manufacturer

Thoclor Labs (Pty) Ltd

Unit E1 Stellenbosch Agri-Park, Baden Powell Drive, Lynedoch, Stellenbosch 7600, South Africa

Bar code No. 6009900445847

Nappi 1179193001

SAHPRA 733MD

Medical Device Class C (ZA)

ISO 13485:2016

For more information on specific applications please visit [www.trifectiv.com/baby](http://www.trifectiv.com/baby) or scan the QR code



\*Each dressing of 3-5 layers of woven gauze requires 5 mL of product – this equates to 50 pumps of the 100 mL bottle per dressing. This ensures optimal wetting of the gauze for healing. This is approx. 20 dressings per bottle.